bulbitech

Every Eye Movement Matters

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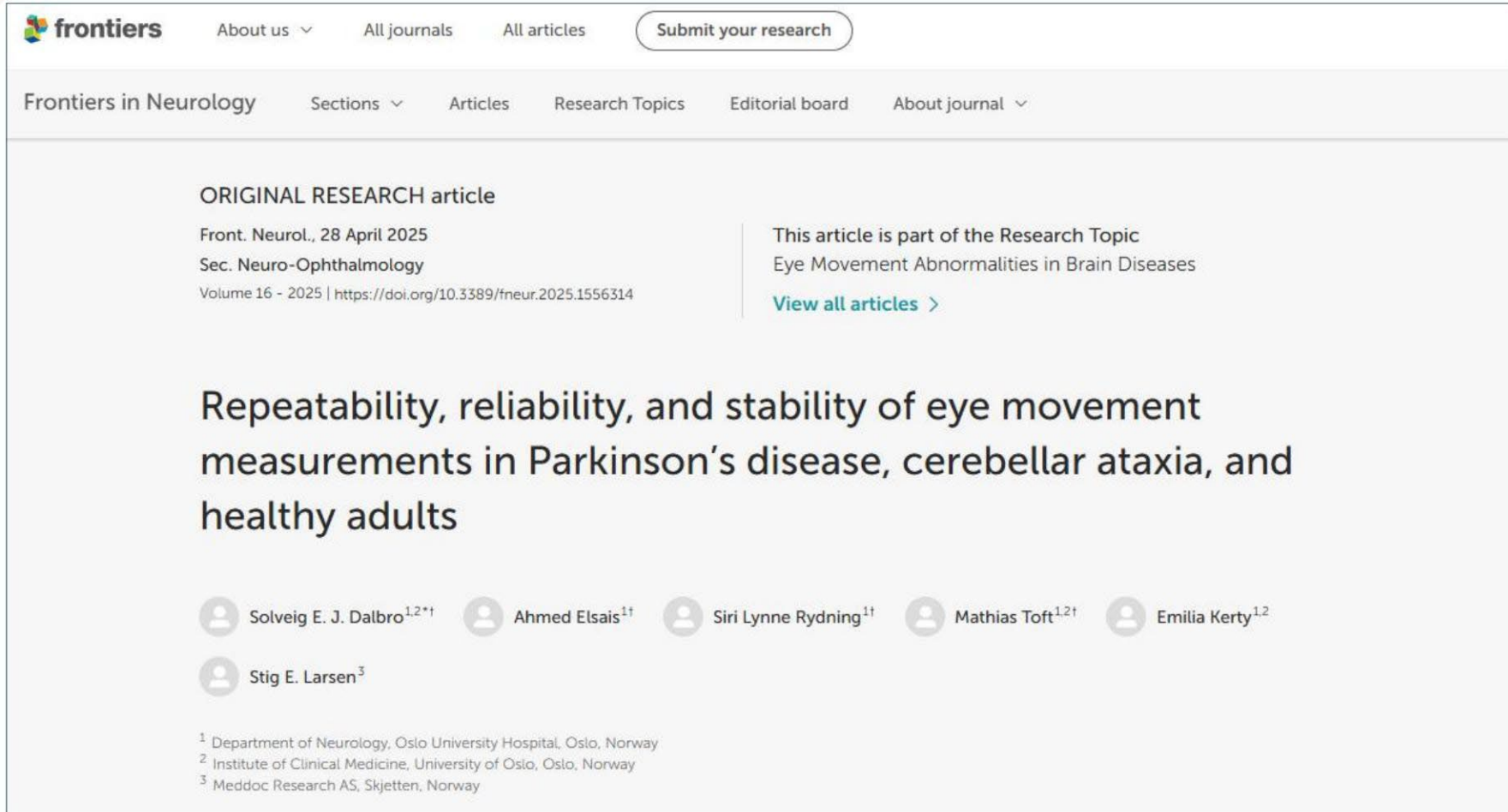
The screenshot shows the top portion of a research article page on the Frontiers website. At the top left is the 'frontiers' logo. To its right are navigation links: 'About us', 'All journals', 'All articles', and a 'Submit your research' button. Below this is a secondary navigation bar for 'Frontiers in Neurology' with links for 'Sections', 'Articles', 'Research Topics', 'Editorial board', and 'About journal'. The main content area features the article type 'ORIGINAL RESEARCH article', publication details ('Front. Neurol., 28 April 2025', 'Sec. Neuro-Ophthalmology', 'Volume 16 - 2025 | https://doi.org/10.3389/fneur.2025.1556314'), and a note that the article is part of a research topic on 'Eye Movement Abnormalities in Brain Diseases' with a 'View all articles' link. The title of the article is 'Repeatability, reliability, and stability of eye movement measurements in Parkinson's disease, cerebellar ataxia, and healthy adults'. Below the title are the authors' names with their corresponding ORCID icons: Solveig E. J. Dalbro, Ahmed Elsaï, Siri Lynne Rydning, Mathias Toft, Emilia Kerty, and Stig E. Larsen. At the bottom, there are three footnotes providing institutional affiliations for the authors.

frontiers About us All journals All articles Submit your research

Frontiers in Neurology Sections Articles Research Topics Editorial board About journal

ORIGINAL RESEARCH article

Front. Neurol., 28 April 2025
Sec. Neuro-Ophthalmology
Volume 16 - 2025 | <https://doi.org/10.3389/fneur.2025.1556314>

This article is part of the Research Topic
Eye Movement Abnormalities in Brain Diseases
[View all articles >](#)

Repeatability, reliability, and stability of eye movement measurements in Parkinson's disease, cerebellar ataxia, and healthy adults

 Solveig E. J. Dalbro^{1,2*}  Ahmed Elsaï^{1†}  Siri Lynne Rydning^{1†}  Mathias Toft^{1,2†}  Emilia Kerty^{1,2}

 Stig E. Larsen³

¹ Department of Neurology, Oslo University Hospital, Oslo, Norway
² Institute of Clinical Medicine, University of Oslo, Oslo, Norway
³ Meddoc Research AS, Skjetten, Norway

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
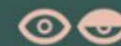
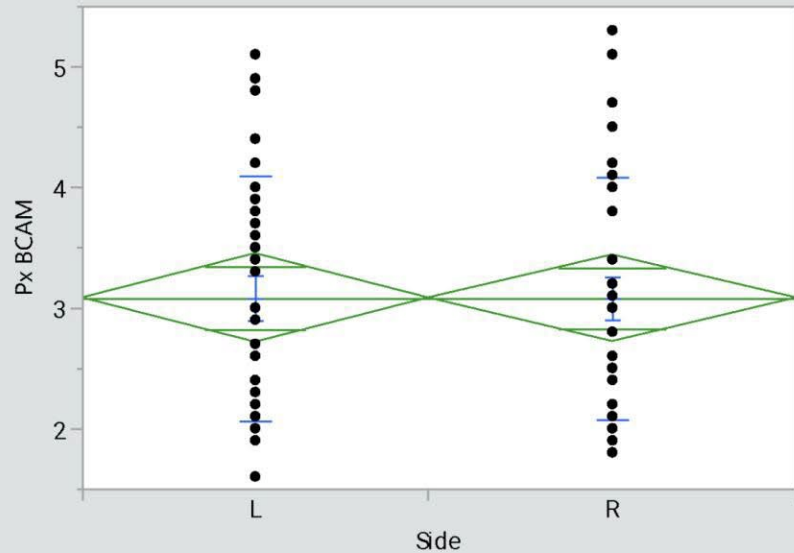
Analysis of pupil diameter between "Left" and "Right" eye for NPX and BCAM

Analysis of Px (pupil diameter mm) BCAM By left and right eye

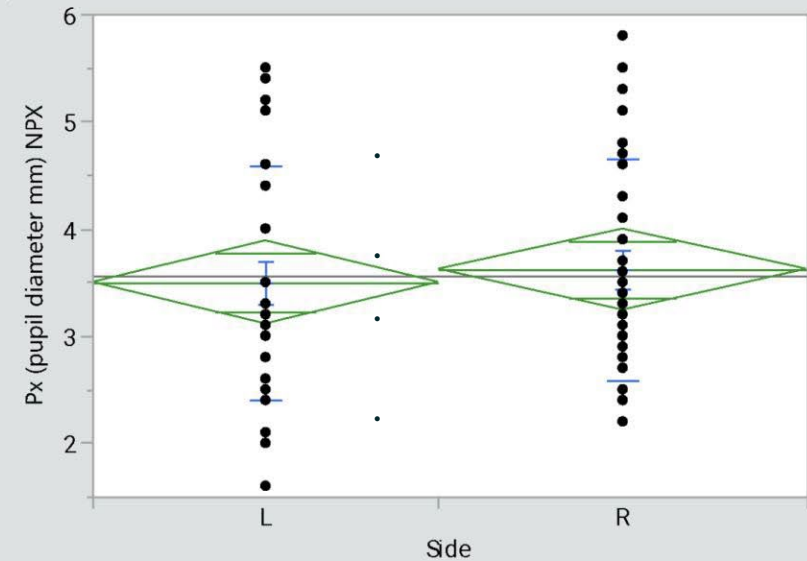Analysis Px (pupil diameter mm) NPi By Side left and right eye

Conclusion:

Small variability between L and R in NPi. Non in BCAM

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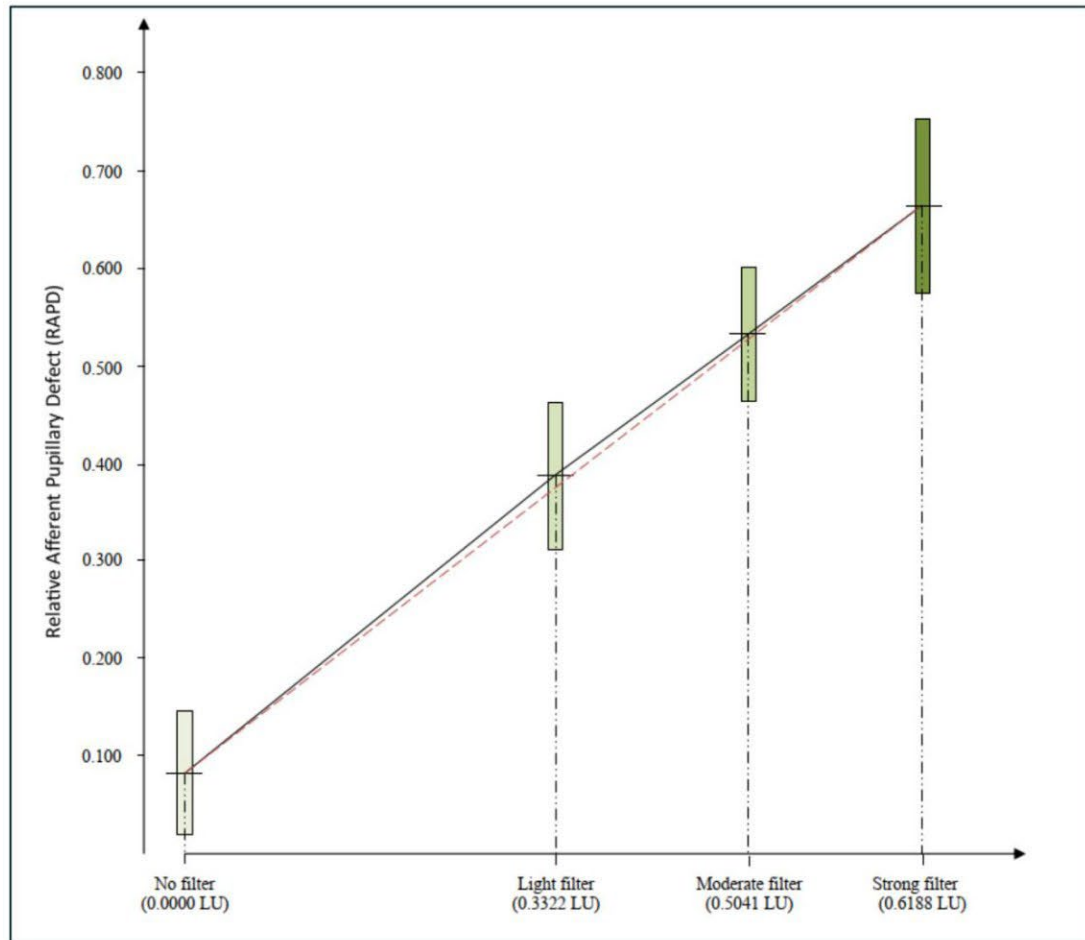
Relative Afferent Pupillary Defect (RAPD) in relation to filter strength of Neutral Density Filters

20 healthy volunteers

Development of Relative Afferent Pupillary Defect (RAPD) in relation to filter strength. The results are expressed as mean values (crossline) with 95% Confidence Intervals (boxes) and full lines between boxes.

The red dotted line indicates linear relationship between "No filter" and "strong filter."

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Case study:
BulbiCAM for diagnosis of Seesaw Nystagmus and monitoring drug efficacy

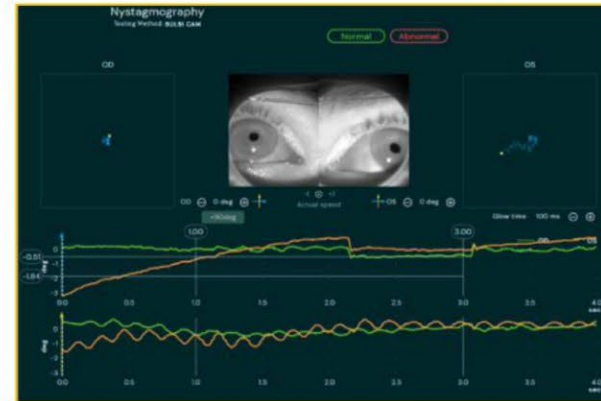
University Medical Center Groningen


Professor Jan Willem Pott:
❖ Head of Neuro-ophthalmology UMCG

Eye Motility Examination: There was a pronounced exotropia in the left eye, but ductions were otherwise within normal limits. A subtle high-frequency vertical nystagmus with a rotary component was evident. Eye movement recordings via the infrared eye-tracker, BulbiCAM, confirmed the presence of a see-saw nystagmus and quantified frequency and amplitude

Management: The patient was initiated on clonazepam at a dosage of 0.5 mg b.i.d.

BulbiCAM screenshot

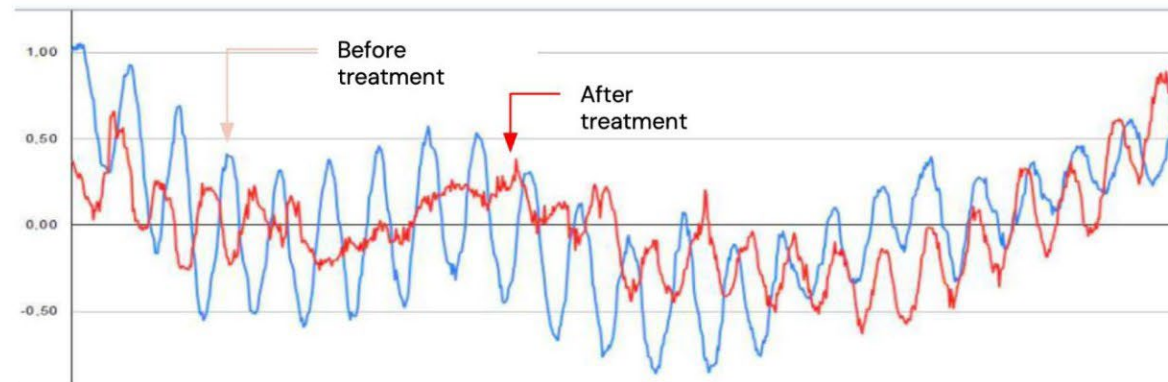Differential amplitude of Seesaw Nystagmus by subtracting vertical nystagmus amplitude of right and left eye. The maximal amplitude of the Seesaw Nystagmus has decreased significantly by treatment with clonazepam.

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Eye-tracking dynamic foveal cone dark adaptation and other function tests stage dry AMD and accurately distinguish from healthy controls

Bjørn André Helland-Hansen, MD, Alexander Sverstad, MD, Goran Petrovski, MD, PhD, Stig Einride Larsen, MD, PhD

Introduction

Background:

- Age-related macular degeneration (AMD) is the leading cause of vision loss in adults over 75.
- Early detection is crucial for timely intervention.
- Traditional diagnostic methods often miss early functional deficits.

Purpose of the Study:

- To validate novel foveal function tests using the BubiCam device.
- Assess the tests' validity, reliability, repeatability, and stability.
- Identify robust biomarkers for early and intermediate dry AMD.

Methods

Study Design:

- Controlled, non-randomised parallel group design.
- Participants: 17 patients with early/moderate dry AMD and 17 age-matched healthy controls (HCs).

Equipment (BubiCAM system):

- Utilizes eye-tracking technology to assess dynamic visual functions.
- Tests conducted: ACOLAPT (Acuity, Contrast, Light Adaptation).

Tests Performed:

- Foveal Resolution Limit (FRL): Measures ability to resolve fine details under dynamic conditions.
- Foveal Contrast Sensitivity (FCS): Assesses contrast detection ability.
- Fixed Frequency-Variable Contrast (Fixfreq_Varcont): Evaluates adaptation to temporal contrast changes.
- Variable Frequency-Fixed Contrast (Varfreq_Fixcont): Tests adaptation to increasing strobe frequencies.
- ACOLAPT ACS: Calculates the relative impact of strobing on contrast sensitivity.

Statistical Analysis:

- Analysis of Variance (ANOVA) and Receiver Operating Characteristic (ROC) analyses.
- Reliability assessed using Intraclass Correlation Coefficient (ICC).
- Repeatability evaluated with Agreement Index (AI) and Stability Index (SI).

Results

Discrimination Between AMD Patients and HCs:

All five BCAM variables significantly differentiated AMD patients from HCs in the worst eye ($p < 0.05$).

Reliability and Repeatability:

Worst Eye	Variable	ICC	AI	SI	Recommended user area
Foveal Eye	Foveal Resolution Limit (FRL)	+	+	+	Population + Patient
	Foveal Contrast Sensitivity (FCS)	+	+	+	Population + Patient
	Fixed Frequency_Variable contrast	+	+	+	Population + Patient
	Variable Frequency_Fixed contrast	+	+	+	Population + Patient**
	ACS	+	+	+	Population + Patient**
Best Eye	Foveal Resolution Limit (FRL)	+	+	+	Population + Patient
	Foveal Contrast Sensitivity (FCS)	+	+	+	Population + Patient
	Fixed Frequency_Variable contrast	+	+	+	Population + Patient
	Variable Frequency_Fixed contrast	+	+	+	Population + Patient**
	ACS	-	-	-	Negative

Eye	Variable	AMD patients	Health controls	AMD/HC Mean (95% CI)	ROC Analysis
Worst Eye	Foveal Resolution Limit (FRL)	6.63(6.02)	9.69(9.02)	8.16(6.64-9.27)	0.79
	Foveal Contrast Sensitivity (FCS)	1.70(0.61)	1.68(0.51)	-1.16(-2.12-0.75)	0.78
	Fixed Frequency_Variable contrast	1.11(0.94)	1.32(0.94)	-2.21(-3.1-0.69)	0.75
	Variable Frequency_Fixed contrast	1.43(0.31)	1.68(0.31)	-1.70(-2.68-0.61)	0.78
	ACS	0.26(0.02)	0.28(0.02)	0.06(-0.01-0.13)	0.79

Table 1: Comparison of mean scores between AMD patients and HCs for each test.

Eye	Variable	Day 1 Mean(SD)	Day 2 Mean(SD)	Day 1 - Day 2 Mean(95% CI)	ICC	AI
Worst Eye	Foveal Resolution Limit (FRL)	6.65 (0.61)	6.62 (0.61)	-0.01 (-1.13-0.96)	0.83	0.75
	Foveal Contrast Sensitivity (FCS)	1.51 (0.61)	1.51 (0.61)	0.00 (-0.16-0.15)	0.77	0.80
	Fixed Frequency_Variable contrast	1.11 (0.94)	1.12 (0.94)	0.01 (-0.18-0.19)	0.86	0.89
	Variable Frequency_Fixed contrast	1.51 (0.31)	1.68 (0.31)	-0.17 (-0.11-0.19)	0.81	0.79
	ACS	0.27 (0.01)	0.28 (0.01)	-0.01 (-0.09-0.07)	0.78	0.23

Recommended user area as BubiCam Acloapt Biomarkers for AMD

Table 2: ICC, AI, and SI values for each variable.

Discussion

Interpretation of Results:

- FRL, FCS, and Fixfreq_Varcont are reliable biomarkers for AMD.
- These tests assess dynamic visual functions often missed by static assessments.
- Varfreq_Fixcont and ACS show potential but require repeated testing for reliability.

Clinical Implications:

- Early detection of functional impairments allows for timely intervention.
- ACOLAPT can be integrated into clinical practice due to its automation and ease of use.
- Dynamic visual function tests provide a more comprehensive assessment of foveal function.

Conclusion

Main Takeaways:

- Validated Biomarkers: FRL, FCS, and Fixfreq_Varcont are effective in distinguishing early AMD from healthy eyes.
- Clinical Utility: ACOLAPT's automated, dynamic testing addresses limitations of traditional methods.
- Future Applications: Potential for early detection and monitoring of AMD progression in clinical settings.

Translational Relevance:

- ACOLAPT could become a valuable tool for clinicians in early AMD detection and patient monitoring.

References

- Wu Z, Aydin LN, Saper BR, Liu CC: Vision function testing for early detection and monitoring of age-related macular degeneration. *Ophthalmol Pract Opt*. 2016;16(1):98-107.
- Brown PE, Lachance RH, Miller J: Light adaptation in cone-mediated foveal function: implications for clinical testing. *Invest Ophthalmol Vis Sci*. 2007;48(7):3288-3293.
- Drexler PA, et al: Measuring rod and cone dynamics in age-related maculopathy. *Invest Ophthalmol Vis Sci*. 2008;49(2):558-65.

Acknowledgments

- Funding Sources: Norwegian Research Council
- Clinical Trial Registration: ClinicalTrials.gov (NCT 05441072), EudraCT (2021-006258-30).

Contact Information

- Bjørn André Helland-Hansen, MD
- Email: bjornahh@ous-hf.no
- Affiliation: Centre for Eye Research, Oslo University Hospital Ullevål, Oslo, Norway

VR system architecture

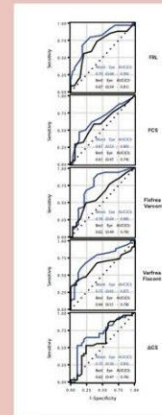
Figure 1: ROC curves demonstrating the discriminative capacity of each BCAM variable.

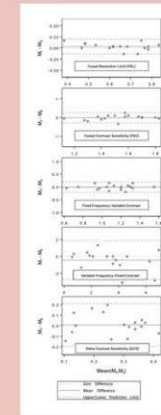
Figure 2: Bland-Altman plots showing agreement between repeated measurements.

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Formation minimale
requis

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World Health Organization | Health Topics ▾ | Countries ▾ | Newsroom ▾ | Emergencies ▾

Over 1 in 3 people affected by neurological conditions, the leading cause of illness and disability worldwide

14 March 2024 | News release | Geneva, Switzerland | Reading time: 3 min (683 words)

A [major new study](#) released by *The Lancet Neurology* shows that, in 2021, more than 3 billion people worldwide were living with a neurological condition. The World Health Organization (WHO) contributed to the analysis of the Global Burden of Disease, Injuries, and Risk Factor Study (GBD) 2021 data.

Neurological conditions are now the leading cause of ill health and disability worldwide. The overall amount of disability, illness and premature death (known as disability-adjusted life years, DALYs) caused by neurological conditions has increased by 18% since 1990.

Over 80% of neurological deaths and health loss occur in low- and middle-income countries, and access to treatment varies widely: high-income countries have up to 70 times more neurological professionals per 100 000 people than low- and middle-income countries.

medscape.com/viewarticle/996637?form=fpf

News > Medscape Medical News

Is the US Neurologist Shortage Insurmountable?

Kelli Whitlock Burton
September 20, 2023

12
29

A national shortage of neurologists is creating "neurology deserts" around the country, forcing patients to wait longer and drive further for care, news that comes as no revelation to Thomas Vidic, MD, clinical associate professor of neurology at the Indiana University School of Medicine-South Bend.

Dr Thomas Vidic

In 2013, Vidic and other members of an American Academy of Neurology (AAN) Workforce Task Force co-authored a [report](#) that predicted the demand for neurologists would outstrip supply by 2025. A decade later, it appears the situation is even more dire than anticipated.

While a nationwide physician shortage is affecting all specialties, neurology is facing a particularly difficult confluence of events. Advances in treatments for migraine, epilepsy, multiple sclerosis, and other neurological disorders have created a growing demand for care of pediatric and adult patients.

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